
AlcoTrx

FCA30

USER MANUAL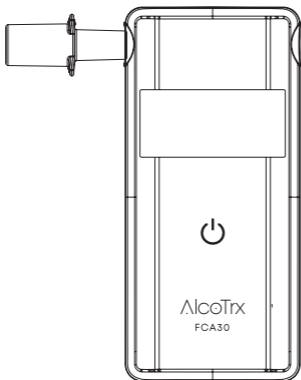

FCA30 user manual

1. ENG
2. SWE
3. NOR
4. FIN
5. GER

AlcoTrx

FCA30

Instruction Manual

- Fuel Cell Sensor
- 4-digit LCD display
- Low battery warning
- Automatic power off
- Total test number display
- Easy to use

EN-16280:2012 APPROVED

Please read the user's manual carefully before use.

Important Safety Information


- Before using the alcohol tester, read all instruction manual carefully and perform the test.
- Recommended to have calibration service every 12 months or 500 tests (whichever comes first) by experts. The use without periodic calibration service may result in invalid result.
- Test should be performed after 30 minutes after eating, drinking or smoking. Failure to wait 30 minutes may result in invalid results or incorrect results.
- Residual alcohol in the mouth can distort the measurement.
- Do not immerse in water.
- Aromatic drinks or mouth sprays containing alcohol can interfere with measurements. In this case, wait for 30 minutes or rinse the mouth with pure water.
- Do not permit to blow cigarette smoke into the tester which may damage the sensor.
- Don't do drunk driving and even small amount of alcohol may cause impairment in normal driving operation.
- Do not use this device as a tool to determine "drink and drive". The manufacturer and the sellers will not take any legal responsibility for the test results.
- When not used for a long time, keep the device after taking out the batteries. In case of testing after long-term storage, there might be high tolerance of test results for the first test.
- Replace the batteries when the battery indicator icon reaches empty for the accuracy reason.
- In case the test result is higher than 0.100 %BAC (1.00 g/L / 0.50 mg/L) and you try again without interval, then the warm up time might be delayed.
- Perform a test within the operating temperature, which is specified in the instruction manual.
- Blowing hard may cause damage to sensor as saliva flows into sensor.

- Do not store the device in the place where the temperature and humidity is high. Recommended to store at room temperature.
- Do not suck in air during the test.
- Recommended to use clean mouthpiece during the test for accurate measurements.
- The device including mouthpieces should be kept out of reach of children under 12 years of age.

Contents

- Digital Breathalyzer
- Two AAA size Batteries
- 3 pieces of mouthpiece
- Instruction manual


Name of each part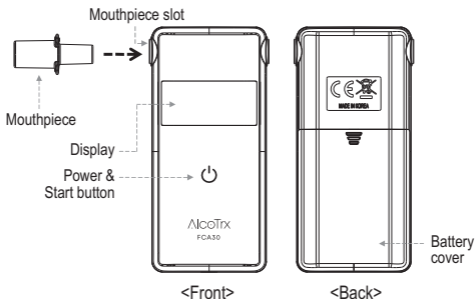

Preparation


1) BATTERY

Insert two batteries in the battery compartment by paying attention to the battery polarity.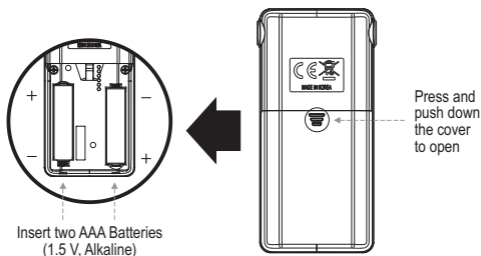

2) MOUTHPIECE

1. Tear the part of plastic bag containing the narrow part of mouthpiece.
2. Hold the part of plastic bag containing the tick part of mouthpiece, and take the narrow part of mouthpiece out of the plastic bag.
3. With holding the tick part of plastic bag, insert the narrow part of mouthpiece into the mouthpiece slot on the side of alcohol tester.
Do not touch the tick part of mouthpiece for the sanitary reasons as it is blowing part.
4. Ensure whether it securely fits or not. It is highly recommended to replace a mouthpiece with new one at each test.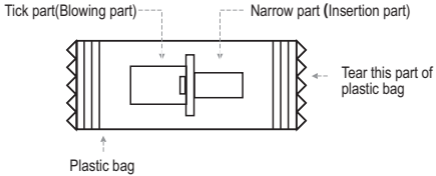

It is highly recommended to use a new mouthpiece for each test for hygienic causes.

The mouthpiece should be discarded at a recycling center after use

Performing the measurement


Step 1. Press the Power button for 1 second. The screen will display the total number of tests performed first, and then the previous test result will be shown.

Note. If you press and hold down the button for more than 5 seconds, the device will be turned off.


Step 2. The countdown will be started automatically.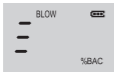

Step 3. When the countdown reaches zero with a double beep sound, "BLOW" will flash and the screen will display three dashes (—) Blow through the mouthpiece for 4 seconds until there is a double beep sound.

Note. The blowing time may be longer or shorter depending on the intensity of blowing.

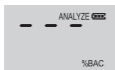Step 4. When the sampling is successful, LCD outer lines will be turning along the edge of LCD as you can see from the picture.

Step 5. The estimated BAC (Blood Alcohol Concentration) will flash for 10 seconds, and the button will be deactivated during this time. After waiting for this time, press power button for 1 second if you want to test again.

- The tester will be turned off automatically in 3 minutes if there is no input signal.
- The forced turn off will be available in Step 2, Step 3 and Step 5. Press and hold power button more than 3 seconds to turn off the tester.

Wait at least 30 minutes after eating, drinking or smoking before measurement.

DRINK DRIVING LIMIT OF EUROPEAN COUNTRIES

0.0 g/l	Czech Republic, Hungary, Romania, Slovakia
0.2 g/l	Cyprus, Estonia, Norway, Poland, Sweden
0.4 g/l	Lithuania
0.5 g/l	Austria, Belgium, Bulgaria, Croatia, Denmark, Finland, France, Germany, Greece, Ireland, Italy, Latvia, Luxembourg, Netherlands, Portugal, Slovenia, Spain, Switzerland
0.8 g/l	Malta, UK

Display message & Trouble shooting guide

MESSAGE	WHAT IT MEANS	ACTION
	The battery is empty.	Replace with new batteries.
BAt	The testing cannot be performed because the battery is low.	Replace with new batteries.
FLO	The sampling volume or blow power is not enough to analyze the breath sample.	Press power button to repeat the countdown cycle and retest.
Out	No breath sample blown within 3 minutes.	Press power button to restart the countdown cycle and retest.
Outc	The testing cannot be performed because the device is turned on outside of the operating temperature.	Repeat the test after the unit has reached a temperature within the operation range (10°C ~ 40°C).
Err2	If the LCD displays "Err2" after Step 4, it means the excessive saliva went into the sensor during the test.	Wait 20 mins, and then try again. If "Err2" is displayed for long time, contact your seller for customer service.
500	The total number of tests has reached 500 times and the calibration service is needed.	Contact your seller for calibration service.
HI	The test results will be shown as HI if the result is over 1.00 ‰ (0.50 mg/L) according to the requirements of EN Standard.	

* The periodic calibration service with every 12 months is highly recommended for maintaining accurate result.

Technical specification

Indication of B.A.C	0.000 ~ 0.100 %BAC 0.00 ~ 1.00 ‰ 0.00 ~ 0.50 mg/L * 0.100 %BAC = 1.00 ‰ = 0.50 mg/L
Accuracy	±0.0075 %BAC(at 0.1 %BAC, at 25°C) ±0.075 ‰(at 1.0 ‰, at 25°C) ±0.0375 mg/L(at 0.50 mg/L, at 25°C)
Warm up time	15 ~ 20 seconds(at 0.05 %BAC, at 25°C) Warm up time may vary depending on the BAC measured.
Sensor type	Fuel cell
Power supply	Two AAA size alkaline batteries
Battery life	Approx. 1000 tests
Dimensions(WxHxD)	51 mm x 109 mm x 18 mm
Weight	84 g (including batteries)
Calibration period	500 tests, or at least every 12 months
Operating temperature	10 ~ 40°C

* The specifications are subject to change without prior notice for functional improvement.

Warranty

The manufacturer warrants the product to be free from defect in workmanship or material (excluding calibration) under normal use either for one year from the date of purchase or 500 breath tests whichever comes earlier.

The manufacturer's obligation under this warranty is limited to replacing, adjusting or repairing the unit at manufacturer's option, if returned along with proof of purchase.

For any defects due to natural disaster or customer's negligence, misuse, reconstruction and repair by unauthorized person, reasonable cost shall be charged for repair or part replacement.

Customer service

For any questions about after-sales service, product calibration or to purchase additional mouthpieces, please kindly contact the nearest distributor or seller. Infor for calibration www.ttmicro.no under service.

Supplier: TT Micro AS / Olaf Helsetsvei 6, 0694 Oslo, Norway
Manufacturer: DA Tech Co., Ltd. / South Korea

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